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Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Instructies

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

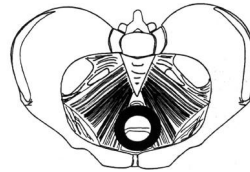
Thick Ring Pessary

Application: The thick ring pessary consists of flexible silicone. In contrast to normal ring pessary it does not include a steel spring filling but consists of a larger circumference of the ring part. This the overlay surface is larger and the risk of necrosis of the vaginal wall may decrease. **Preferably older patients with descensus and mild cases of prolapse without incontinence are treated with this type.** An intact pelvic floor (also if reduced) is required for this pessary.

Sizes: Various sizes of the ring pessary can be adapted, ranging from minimum 50mm to maximum 100mm. The pessary with the smallest diameter that just holds should be prescribed.

Use: The physician adapts the pessary on the first examination including tests like coughing, pressing and movements. For the insertion the pessary is covered with estrogen cream. After the test, the position of the pessary should be checked again.

If possible it should be recommended that the patient changes the pessary herself, i.e. she has to **remove it in the evening and to insert it in the morning**. The coating with estrogen cream facilitates the insertion and improves the local blood circulation as well as the formation of epithelium and connective tissue. Exclusively creams containing estriol (shorter adhesion to the receptor) should be used. Since one needs a relatively small amount of cream for the gliding effect, we recommend to use a concentration of 1 mg estriol/1 g salve.

The change of the pessary may be done while standing whereby the patient may put one foot on a stool or the edge of a bed, in a position comparable to the insertion of a tampon; if this is too difficult, it may be sufficient to insert the pessary while spreading the legs, if necessary while leaning against a wall or while lying. The patient should be trained how to insert the pessary (e.g. first to press it and then push it to the upper part of the vagina) and how to remove it (surrounding of the ring with her forefinger, edging and loosening of the pessary). However, older patients may prefer to be assisted by a nurse at regular intervals. Under these conditions, one should insert an estriol cream or ovulum once a week.

Side Effects/Complications: The therapy with this pessary is meant to cure or at least to reduce the patient's problems caused by prolapse - in combination with additional measures (physiotherapy, estrogens). The daily change by the patient and the additional use of estrogen cream after the menopause avoids complications like infections, bleedings or even ulcers. If the prolapse is insufficiently cured or if various sizes do not stay in their position one has to change to a cube pessary and local estrogen therapy. Contraindications for estriol creams should be considered (e.g. pregnancy, lactation period, estrogen-dependent tumors).

If the patient is not able to change the pessary by herself, it may be advisable to integrate a nurse or a member of the family into the procedures.

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar) or hot air (250°C).

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